



female's Health

FERTILITY TEST FOR WOMEN



LYHER®

With Lyher series products, we monitor women fertility-related hormones that suggest the best time for pregnancy and confirm the pregnancy status in very early stage. We care and we do understand the importance of accurate diagnosis with reliable results in a confidential and convenient method.

HCG Pregnancy Test / LH Ovulation Test / FSH Menopause Test




- Easy to operate
- Fast to read visual result in 3/5 minutes
- Accuracy greater than 99%
- Easy to carry and test at home or on the road



Digital Pregnancy (HCG) Test Pen



With clear indication from the LED screen, you can read the result of pregnancy status without any difficulties.

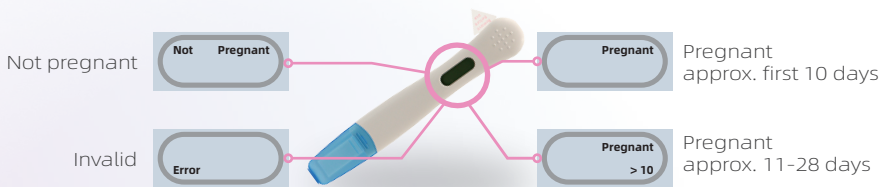

- Easy to operate
- Easy and fast to read intuitive results indicating pregnancy days
- Accuracy greater than 99%
- Language customization available

Digital Ovulation (LH) Monitor



Monitoring and analysis of non-ovulation period, ovulation period, ovulation peak



Scientific pregnancy preparation


Suitable for polycystic, elderly, irregular menstruation and other women

Scheduled pregnancy

Monitoring of ovarian function



LH levels rise and fall, based on where you are in your menstrual cycle. LH “surges” and is at its highest point just before ovulation occurs. After ovulation, LH helps the body get ready for a potential pregnancy, by promoting the production and secretion of progesterone from the corpus luteum.

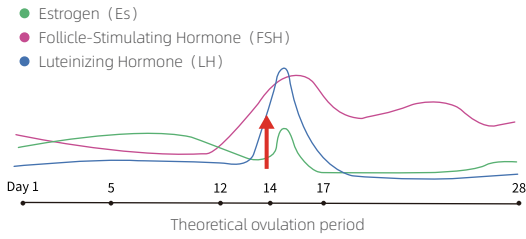

Indicates a LOW result with a low probability of conception. Please test again after 24 hours. To ensure the accuracy of the tests, please keep testing every day.




Indicates HIGH result, when the test detects the rise of LH, "+" will be displayed, which indicates a high probability of conception. Please test every 4-6 hours afterwards. Have intercourse today to increase your chance of getting pregnancy.



Indicates a PEAK result which means an LH surge is detected. And you have the highest probability of conception. Please keep testing every 4-6 hours until it shows "+" or "-", which indicates that you have ovulated. Have intercourse within 24 hours to maximize your chance to be pregnant.

- Easy to operate
- Easy and fast to read intuitive results
- Accuracy greater than 99%
- Ovulation period tracking
- Supportive to upload results to the designated APP, easy to record and demonstrate the hormone (ovulation) change curve
- Reusable, in line with the concept of sustainable development

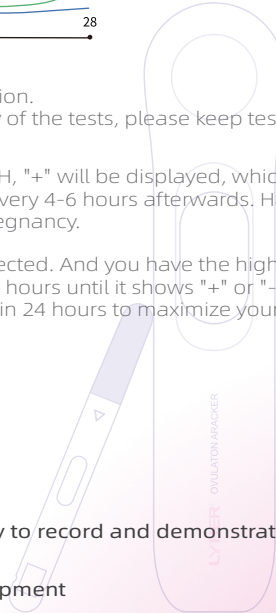

Ordering Information

Product name	Format	Specimen	Test time	Shelf life
HCG Pregnancy Test	Strip	Urine	3 mins	24 months
	Cassette	Urine	3 mins	24 months
	Midstream	Urine	3 mins	24 months
	Digital pen	Urine	5 mins	24 months
LH Ovulation Test	Strip	Urine	5 mins	24 months
	Cassette	Urine	5 mins	24 months
	Midstream	Urine	5 mins	24 months
LH Ovulation Monitor	Digital pen	Urine	5 mins	24 months

EXPECTING A NEW LIFE.

Hangzhou Laihe Biotech Co., Ltd.

 Room 401-406, F1-3, Building 1, No.425 Miaohouwang Road, Xixing Street, Binjiang District, Hangzhou, 310051 Zhejiang P.R. China

 +86-571-87653090

 www.lyherbio.com

 office@lyher.com